

ERYCLONE[®]

ANTI-D (Rho) (IgM)

MONOCLONAL BLOOD TYPING ANTIBODIES FOR SLIDE AND TUBE TESTS

SUMMARY

Monoclonal antibodies are derived from hybridoma cell lines, created by fusing mouse antibody producing B lymphocytes with mouse myeloma cells or are derived from a human B cell line through EBV transformation.

Each hybridoma cell line produces homogenous antibodies of only one immunoglobulin class, which are identical in their chemical structure and immunological activity.

Human red blood cells are classified as Rho (D) positive or Rho (D) negative depending upon the presence or absence of D (Rho) antigen on them. Approximately 85% of the Caucasian population are Rho (D) positive. The D⁺ phenotype is a variant of D (Rho) antigen and is recognised by performing the antiglobulin test.

About 60% of the D⁺s, now classified as weak or partial D's, may react with Anti-D (Rho) (IgM) in slide tests and about 90% may be detected by the tube technique.

REAGENT

ERYCLONE[®] Anti-D (Rho) (IgM) is a ready to use reagent, prepared from supernatants of cell cultures with antibody producing B lymphocytes obtained through EBV transformation and is a blend of Agglutinating sera of immunoglobulin class IgM (Clone P3x61+NaTH119), having the capability of recognising different epitopes of the human red blood cell antigen D (Rho).

ERYCLONE[®] Anti-D (Rho) (IgM) does not detect all weak and partial D's. For the confirmation of negative reactions with **ERYCLONE[®]** Anti-D (Rho) (IgM) further testing with an incomplete Anti-D (Rho) of IgG class or **RHOFINAL[®]** Anti-D (Rho) (IgM+IgG) (REF: 101660010 & 101660610) is strongly recommended to confirm the presence or absence of weak/partial D's.

Each batch of reagent undergoes rigorous quality control at various stages of manufacture for its specificity, avidity and performance.

REAGENT STORAGE AND STABILITY

1. Store the reagent at 2-8°C. DO NOT FREEZE.
2. The shelf life of the reagent is as per the expiry date mentioned on the reagent vial label. Once opened the shelf life of the reagent vial is as described on the reagent vial label provided it is not contaminated.

PRINCIPLE

Human red blood cells possessing the D (Rho) antigen will agglutinate in the presence of the Agglutinating sera directed towards the antigen. Agglutination of red blood cells with **ERYCLONE[®]** Anti-D (Rho) (IgM) reagent is a positive test result and indicates the presence of D (Rho) antigen. No agglutination with the reagent is a negative test result and indicates the absence of D (Rho) antigen. All negative test results should be further tested for D⁺ (Presence of weak / partial D's) by performing the D⁺ test procedure using incomplete Anti-D (Rho) of IgG class, as described later.

PRECAUTIONS

1. In vitro diagnostic reagent for laboratory and professional use only. To be used by a qualified personnel. Not for medicinal use.
2. The reagent contains sodium azide 0.1% as preservative. Avoid contact with skin and mucosa. MSDS available on request.
3. Extreme turbidity may indicate microbial contamination or denaturation of protein due to thermal damage. Such reagents should be discarded.
4. Reagents are not from human source, hence contamination due to HBsAg, HIV and HCV is practically excluded.
5. It is necessary to use the dropper provided in the reagent vial to dispense a reagent drop.
6. It is advisable to wear gloves and safety spectacles and handle test specimens of human origin with caution.
7. Do not use damaged or leaking reagents.
8. Special protective measures, conditions for disposal and disinfection should be implemented in accordance with local regulations.

SAMPLE COLLECTION AND PREPARATION

No special preparation of the patient is required prior to sample collection by approved techniques. Sample should be stored at 2-8°C if not tested immediately. For optimal results, freshly collected sample should be used. Anticoagulants like EDTA, CPD-A and Citrate can be used. Do not use haemolysed samples.

ADDITIONAL MATERIAL REQUIRED FOR SLIDE AND TUBE TESTS

Slides (60 x 85 mm), Test tubes (12 x 75 mm), Micropipettes, Isotonic saline (0.9% NaCl), Centrifuge, Timer, Mixing sticks, **ERYCLONE[®]** Anti-D (Rho)(IgG) or **RHOFINAL[®]** Anti-D (Rho) (IgM+IgG) and **ERYCLONE[®]** Anti-Human Globulin (Coombs) reagent.

TEST PROCEDURE

Bring reagent and samples to room temperature before testing.

Slide Test

1. Place one drop of **ERYCLONE[®]** Anti-D (Rho) (IgM) reagent on a clean slide.
2. Pipette 50µl of whole blood on the slide.
3. Mix well with a mixing stick uniformly over an area of approximately 2.5 cm².
4. Rock the slide gently, back and forth.
5. Observe for agglutination macroscopically at the end of two minutes.

Immediate Spin Tube Test

1. Prepare a 5% suspension of red cells to be tested in isotonic saline.
2. Place one drop of **ERYCLONE[®]** Anti-D (Rho) (IgM) reagent into a labeled test tube.
3. Pipette into the test tube 1 drop (~50µl) of 5% cell suspension and mix well.
4. Centrifuge for one minute at 125 g or 20 seconds at 1000 g or alternative speed and time validated by the laboratory.
5. Gently resuspend the cell button, observing for agglutination macroscopically.

Microplate Method

ERYCLONE[®] reagents are standardized for use in Microplate technique, however it is recommended that each laboratory should standardize and validate their own procedure using "U" bottom microplates. The below mentioned procedure is a guideline which should be considered while standardization and validation process of procedure because of the variation in methods and equipment used in various laboratories.

1. Prepare 2-3% suspension of the red cells to be tested in isotonic saline.
2. Add 1 volume of **ERYCLONE**® Anti-D (IgM) reagent using the reagent vial dropper into appropriate microwells.
3. Add equal volume of test red cell suspension.
4. Mix the contents of the microplate taking care to avoid cross-well contamination.
5. Centrifuge the microplate at low spin (400g for 30 secs). Or Centrifuge at a speed and time as per the standardization and validation process done using same centrifuge.
6. Re-suspend the red cells using a microplate shaker or manually.
7. Read and record results.

Note: Incubation of 10-15 mins at Room Temperature may be included to enhance the reactivity.

D⁺TEST PROCEDURE

1. Prepare a 5% suspension of the red cells to be tested in isotonic saline.
2. Place one drop of any incomplete Anti-D (IgG) such as: **ERYCLONE**® Anti-D (Rho) (IgG class) reagent or **RHOFINAL**® Anti-D (Rho) (IgM+IgG) (REF: 101660010 & 101660610) into a labeled test tube.
3. Add to the test tube 50µl of the 5% cell suspension and mix well. Incubate at 37°C for 15 minutes.
4. Wash the contents of the tube thoroughly, atleast three times, with isotonic saline and decant completely after the last wash.
5. Add 100µl of **ERYCLONE**® Anti Human Globulin reagent and mix well.
6. Centrifuge for 1 minute at 1000 RPM (125g) or 20 seconds at 3400 RPM (1000g).
7. Very gently, resuspend the cell button and observe for agglutination macroscopically.

INTERPRETATION OF RESULTS

Slide and Tube Tests

- a) Agglutination is a positive test result and indicates the presence of D (Rho) antigen. Do not interpret peripheral drying or fibrin strands as agglutination. No agglutination is a negative test result and indicates the absence of D (Rho) antigen.
- b) Cord cells heavily sensitized with **ERYCLONE**® Anti-D (Rho) may give a false negative immediate spin test result.
- c) It is strongly recommended that as a routine quality control measure known as Rho (D) positive and Rho (D) negative red cells be occasionally run, preferably on a daily basis so as to control reagent performance and validation of test results.

D⁺Test Procedure

(a) Agglutination with reagent indicates the presence of D⁺ antigen (weak / partial D's). (b) No agglutination with reagent indicates the absence of D⁺ antigen (Absence of weak / partial D's). Negative reactions obtained in D⁺ test must be validated:- add 50µl of Coomb's control cells to the reaction mixture. A positive reaction confirms the activity of the Coomb's reagent and validates the negative reaction before the addition of the Coomb's control cells. (c) Mixed field agglutination in the D⁺ test on red cells from a recently delivered woman may indicate a mixture of maternal Rho (D) negative and fetal Rho (D) positive blood. (d) Red cells demonstrating a positive direct antiglobulin test cannot be accurately tested for D⁺ antigen (Presence of weak / partial D's).

Note

1. It is strongly recommended that red cells with known Rh characteristics should be periodically run, preferably on a daily basis to validate the reagent performance.
2. The quality control of reagents is performed by incubating at 37°C for 30 minutes for Anti-D IgM reagent.
3. The reagents are expected to exceed minimum specifications/acceptance criteria as per label claim for titre, specificity & avidity as laid down by transfusion medicine technical manual- 2003 with reference to NIBSC-UK & WHO international ; standards.

REMARKS

1. As undercentrifugation and overcentrifugation could lead to erroneous results, it is recommended that each laboratory calibrate its own equipment and the time required for achieving the desired results.
2. After usage, the reagents should be immediately recapped and replaced to 2-8°C storage.

PERFORMANCE CHARACTERISTICS

The performance of **ERYCLONE**® Anti-D (Rho) (IgM) complies with the common technical specifications of in-vitro diagnostic medical devices under the recommended methods.

The performance of **ERYCLONE**® Anti-D (Rho) (IgM) was evaluated on over 3275 samples (from donors, clinical and neonates) drawn in the recommended anticoagulants. The process, techniques and protocols used were as defined in the package insert. The evaluation demonstrated 100% specificity. The sensitivity of reagent by slide test is 98.77% and by tube test it is 99.37% versus the expected results with common known Rhesus phenotypes.

WARRANTY

This product is designed to perform as described on the label and package insert. The manufacturer disclaims any implied warranty of use and sale for any other purpose.

BIBLIOGRAPHY

1. Kohler C. & Milstein C. (1975), Continuous cultures of fused cells secreting antibody of predefined specificity., Nature, 256, 495-497.
2. Lee H.H., Rouger P., Germain C., Muller A & Salmon C. (1983). The production and standardisation of monoclonal antibodies as AB blood group typing reagents. Symposium of International Association of Biological Standardisation on Monoclonal antibodies.
3. Human Blood Groups by Geoff Daniels, 1st Ed., Blackwell Science, Oxford 1995.
4. HMSO, Guidelines for Blood Transfusion Services., 2nd Ed., 1994.
5. Data on file: Tulip Diagnostics (P) Ltd.

Manufactured by:

 **TULIP DIAGNOSTICS (P) LTD.**

REGD. OFFICE: GITANJALI, TULIP BLOCK, DR. ANTONIO DO REGO BAGH, ALTO SANTACRUZ, BAMBOLIM COMPLEX P.O., GOA-403 202, INDIA. Website: www.tulipgroup.com E-mail: sales@tulipgroup.com

MANUFACTURING UNIT: PLOT NOS. 92/96, PHASE II C, VERNA IND. EST., VERNA, GOA-403 722, INDIA.

In vitro Diagnostic Reagent
NOT FOR MEDICINAL USE.
Store at 2-8°C. DO NOT FREEZE.
Preservative: 0.1% NaN₃
AN ISO 13485 Certified Company

01680625WER-03
ERYCLONE Anti-D (Rho) (IgM)